Prior to use, carefully read the instructions for use. **EN**

# CERASMART™

FORCE ABSORBING HYBRID CERAMIC CAD/CAM BLOCK

For use only by a dental professional in the recommended indications.

## INDICATIONS

1. Metal free indirect restorations: full crown, inlay, onlay, laminated veneer
2. Implant supported crown


## CONTRAINDICATIONS

1. Avoid use of this product in patients with known allergies to methacrylate polymer.

## PREPARATION DESIGN

When designing restorations, the following minimum dimensions should be maintained:

- Minimum wall thickness should be 1.5 mm, 1.0 mm at margins.
- Prepare margins with deep chamfer or rounded shoulder. (Figure 1)
- For inlays and onlays, all internal edges and angles should be rounded. Avoid having margins in direct occlusal contact with the opposing tooth.
- Minimum thickness of the restoration should be 1.5 mm in pit and fissure areas and 1.5 mm in cusp areas.

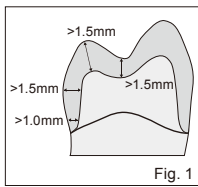## MILLING

Please refer to respective instructions of the milling unit. Select the milling program for CERASMART or composite resin block materials. After milling, check the restoration for:
 

- Discolorations linked to the milling process
- Protuberances
- Cracks or chips

If the restoration is defective, discontinue the manufacture of restoration.

## FINISHING AND POLISHING

Finish and polish with coarse and fine silicone point. DIAPOLISHER PASTE can be used for final polishing to obtain high gloss.

## NOTE:

When adjusting or polishing the restorative, use a dust collector and wear a dust mask to avoid inhaling dust. Wear protective glasses.

## CEMENTATION

- 1) Sandblasting technique
- 2) Blow the restoration with 25-50µm alumina 0.15MPa/1.5bar is recommended.
- 3) Treat the restoration with an oil-free air syringe or clean with an ultrasonic cleaner and dry.
- 4) Clean with alcohol to remove oil residue.
- 5) Treat the surface with a silane coupling agent such as CERAMIC PRIMER II or G-Multi PRIMER.
- 6) Cement with an adhesive resin cement such as G-CEM LinkForce.

NOTE: Prior to using CERAMIC PRIMER II, G-Multi PRIMER or G-CEM LinkForce, refer to the respective instructions for use of the manufacturer.

- 1) Treat with hydrofluoric acid (5%) for 60 seconds.
- 2) Wash with water spray or an ultrasonic cleaner and dry.
- 3) Treat the surface with a silane coupling agent such as CERAMIC PRIMER II or G-Multi PRIMER.
- 4) Cement with an adhesive resin cement such as G-CEM LinkForce.

NOTE: Prior to using CERAMIC PRIMER II, G-Multi PRIMER or G-CEM LinkForce, refer to the respective instructions for use of the manufacturer.

- 1) Hydrofluoric acid is for extraoral use only and must not be applied in the oral cavity.
- 2) Phosphoric acid (35-37%) can be also used for the purpose of cleaning the surface, preferably scrubbing with a microbrush for 10-15 seconds.

## ADDITIONAL APPLICATION AND REPAIR

Roughen the bonding area using diamond point or carbide bur. Apply CERAMIC PRIMER II or G-Multi PRIMER to improve adhesion and dry the surface. Apply a bonding agent following its manufacturer's instruction. Proceed with the application of the chosen composite restorative material.

G-aenial Universal Flo is recommended in order to make the surface of restorative homogeneous as it consists of the same flexible nano ceramic technology as CERASMART. In this case, the intermediate use of a bonding agent is not necessary. Alternatively, OPTIGLAZE color is also recommended as surface characterization and glazing technique. Sandblast with 25-50µm alumina (0.15MPa/1.5bar), clean and dry. Apply CERAMIC PRIMER II or G-Multi PRIMER, don't apply OPTIGLAZE color. OPTIGLAZE color can only be cured with a light-curing device with a wavelength of 400-430nm. Refer to the respective instructions for use.

## SHADE

- 5 Vita shades\* available in two translucencies:
  - Low Translucency (HT): A1 HT, A2 HT, A3 HT, A3.5 HT, B1 HT
  - High Translucency (LT): A1 LT, A2 LT, A3 LT, A3.5 LT, B1 LT

\*One bleaching shade should always be worn.

\*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

## SIZE

Available in 3 sizes: 12, 14, 14L.

## PACKAGE

5 blocks per box

## STORAGE

Recommended for optimal performance, store at room temperature (4-25°C / 39.2-77.0°F) away from direct sunlight and high humidity.

## CAUTION

1. In rare cases this product may cause sensitivity in some people. If such reactions are experienced, discontinue the use of product and consult a physician.
  2. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for the Americas <http://www.gcamerica.com>. They can also be obtained from your supplier.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental). In this way you will contribute to improve the safety of this product.

Last revised: 07/2019

MANUFACTURED BY:  
GC DENTAL PRODUCTS CORP.  
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan  
RESPONSIBLE MANUFACTURER IN CANADA:  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
DISTRIBUTED BY:  
GC CORPORATION  
70-1 Hamaumachi-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU** : GC EUROPE N.V.  
Rue de l'Industrie, 1260, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00  
**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1 708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

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