Prior to use, carefully read the instructions for use.

COCOA BUTTER

PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24hours after placement.
- To prevent cement from sticking to adjacent teeth and instruments.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to cocoa butter or nut.

HOW TO USE

- Place glass ionomer cement according to the manufacturer's instructions.
- Extrude the required amount of material from the tube. Using a spatula or cotton pellet, apply uniformly to the area to be moisture proofed / isolated.
- After use, immediately replace cap.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

PACKAGE

Tube of 10 g
Tube of 2 g

(All packaging types may not be available in all countries.)

CAUTION

- In case of contact with eyes, flush immediately with water and seek medical attention.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERSER. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

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