EQQUIA

SELF-ADHESIVE POSTERIOR RESTORATIVE SYSTEM

***GC***

# EQUIA™ Fil / EQUIA™ Coat

|    |   |     |
|----|---|-----|
| EN | BULK FILLED RADIOPAQUE POSTERIOR RESTORATIVE IN CAPSULES / LIGHT CURED SELF ADHESIVE WEAR RESISTANT COATING   | 3   |
| DE | BULKAPPLIZIERBARES, RÖNTGENSICHTBARES SEITENZAHNRESTAURATIONS-MATERIAL IN KAPSELN / LICHTHÄRTENDES, SELBSTADHÄSIVES, ABRASIONSBESTÄNDIGES COATING                         | 13  |
| FR | MATERIAU DE RESTAURATION POSTERIEUR CONDENSABLE EN CAPSULES / TRAITEMENT DE SURFACE AUTO-ADHESIF RESISTANT A L'USURE PHOTOPOLYMERISABLE                                   | 23  |
| IT | MATERIALE RADIOPACO IN CAPSULE PER RESTAURI POSTERIORI IN UNICO APPORTO / RIVESTIMENTO AUTOADESIVO, FOTOPOLIMERIZZABILE, RESISTENTE ALL' USURA                            | 33  |
| ES | RESTAURADOR DE RELLENO EN BLOQUE PARA POSTERIORES, RADIOPACO Y PRESENTADO EN CÁPSULAS / CAPA FOTOPOLIMERIZABLE Y AUTOADHESIVA RESISTENTE A LA ABRASIÓN                    | 43  |
| NL | IN BULK TE APPLICEREN RADIOPAAK POSTERIEUR RESTAURATIEMATERIAAL IN CAPSULES / LICHTHARDENDE ZELFHECHTENDE SLUTVASTE COATING   | 53  |
| DA | RADIOPAQUE POSTERIOR T BULKFYLDNINGSMATERIALE I KAPSLER / LYSHÆRDENDE OG SELVKLÆBENDE SLIDSTÆRK COATING   | 63  |
| SV | BULK APPLICERING, RÖNTGENKONTRAST, POSTERIOR T FYLLNINGSMATERIAL I KAPSLAR / LJUSHÄRDANDE SJÄLVETSANDE NÖTNINGSRESISTENT COATING  | 73  |
| PT | MATERIAL RESTAURADOR RADIOPACO DE INCREMENTO ÚNICO PARA RESTAURAÇÃO DE DENTES POSTERIORES EM CÁPSULA / REVESTIMENTO AUTOADERENTE FOTOPOLIMERIZÁVEL RESISTENTE AO DESGASTE | 83  |
| EL | ΕΝΙΣΧΥΜΕΝΗ ΑΚΤΙΝΟΣΚΙΕΡΗ ΑΠΟΚΑΤΑΣΤΑΤΙΚΗ ΣΥΝΘΕΤΗ ΡΗΤΙΝΗ ΟΠΙΣΘΙΩΝ ΣΕ ΚΑΨΟΥΛΕΣ / ΦΩΤΟΠΟΛΥΜΕΡΙΖΟΜΕΝΟ ΑΥΤΟΑΔΡΟΠΟΙΟΥΜΕΝΟ ΑΝΘΕΚΤΙΚΟ ΣΤΗΝ ΑΠΟΤΡΙΒΗ ΕΠΙΧΡΙΣΜΑ                       | 93  |
| FI | RÖNTGENOPAAKKI TAKA-ALUEEN BULK FILL -PAIKKA-AINE KAPSELEISSA / VALOKOVETTEINEN ITSESTÄÄNKIINNITTYVÄ KULUTUSTA KESTÄVÄ PINNOITE   | 103 |
| NO | BULK-FILL RADIOPAK GLASSIONOMER Fyllingsmateriale til posterior bruk / LYSHÆRDENDE, SELVBONDENDE OG SLITESTÆRK COATING  | 113 |

Prior to use, carefully read the instructions for use.  
For use only by a dental professional in the recommended indications.

## **RECOMMENDED INDICATIONS**

1. Class I restorations
2. Non-stress bearing Class II restorations
3. Stress bearing Class II restorations when isthmus is less than half of intercuspal distance
4. Intermediate restorative
5. Class V and root surface restorations
6. Core build up

EQUIA Coat is used to seal and protect the surface of EQUIA Fil restorations.

## **CONTRAINDICATIONS**

1. Pulp capping.
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

## **NOT TO BE USED**

EQUIA Coat should not be used in combination with desensitizers and eugenol containing materials as these may hinder EQUIA Coat from setting or bonding properly.

## DIRECTIONS FOR USE

|  |                    |
|--|--------------------|
| Powder / Liquid Ratio (g/g)                                    | 0.40 / 0.12        |
| Mixing Time (sec.)*  | 10"                |
| Time of initial extrusion after mixing                         | 10 seconds maximum |
| Working Time (23 °C/73 °F) (min., sec.)                        | 1' 15"             |
| Net Setting Time (37 °C/99 °F) (min., sec.)                    | 2' 00"             |
| Finishing Time (37 °C/99 °F, from start of mixing) (min.,sec.) | 2' 30"             |
| VLC curing time of EQUIA Coat                                  | 20"                |

## 1. CAVITY PREPARATION

- a) Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide. In class V or II situations, make sure to use a right matrix system. Metal bands should be lubricated with petroleum jelly.
- b) Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge.
- c) Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

## 2. CAPSULE ACTIVATION AND MIXING

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder.
- b) To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds.  
Note: 1) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.  
2) The capsule should be activated just before mixing and used immediately.
- c) Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000 RPM).  
Note: When using a rotational mixing device, it is recommended not to use the centrifuge option.

### 3. RESTORATIVE TECHNIQUE

- a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- b) Make two clicks to prime the capsule then syringe. The working time is 1 minute 15 seconds from start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- c) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation.
- d) Form the preliminary contour, and cover with a matrix if required.
- e) During the first 2 minutes 30 seconds from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply EQUIA Coat and light cure.  
Note: 1) To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.  
2) To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.

### 4. FINISHING

Finish under water spray using superfine diamond burs after 2 minutes 30 seconds from start of mixing.

Note: When applying EQUIA Coat on the existing EQUIA restorations, roughen the surface to be coated with superfine diamond burs.

## 5. COATING

- a) Spray preparation dust away with water. Dry by gently blowing with oil free air. Surfaces to be coated should be dry. Do not desiccate.
- b) Dispense a few drops of EQUIA Coat into a dispensing dish. Replace bottle cap immediately after use.
- c) IMMEDIATELY apply (within 1 minute after dispensing) to the surfaces to be coated using the disposable applicator. Use floss to apply to approximal surface. DO NOT AIR BLOW.

Note: 1) Non bur cut enamel should be treated with a 35-40% phosphoric acid according to the manufacturer's instructions.  
Rinse and dry with oil free air.

- 2) Should the coated surface be contaminated with water, blood or saliva prior to light curing, wash and dry the surface and repeat the coating procedure.

## 6. LIGHT CURING

IMMEDIATELY light cure all coated surfaces with a visible light curing unit ( $> 500\text{mW}/\text{cm}^2$  : Halogen, GC G-Light, LED) for 20 seconds.

Note: 1) Place light source as close as possible to the coated surface.

2) If surface is tacky or yellowish, repeat light curing.

3) Use a protective light shield or similar protective eye wear during light curing.

## STORAGE

Recommended for optimal performance, store the EQUIA Fil capsules in the original aluminium foil in a cool and dark place (4-25°C) (39.2-77.0°F).

If not in use for a prolonged period of time, store EQUIA Coat in refrigerator, otherwise store at room temperature (4-25°C)(39.2-77.0°F).

## SHADES

Available in Refills: A1, A2, A3, A3.5, B1, B2, B3, C4, SW (Standard White)\*

Shade numbers according to Vita® shade guide.

Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

\*Only available in limited countries and packages.

## PACKAGES

1. Single shade package: 50 capsules (A1, A2, A3, A3.5, B1, B2, B3, C4, SW)
2. Assortment package : 50 capsules (10 each of A2, A3, A3.5, B1, B3)

3. Single shade package: 100 capsules (A2, A3)
4. Double shade package: 100 capsules (A2-A3, A3-B2)
5. Single shade package: 250 capsules (A2, A3, B2)  
Average contents per capsule : 0.40g powder and 0.12g (0.10mL) liquid. Minimum net volume of mixed cement per capsule : 0.14mL.
6. EQUIA Coat 4mL (1) / 6mL\* (1), disposable applicator (25).

Option : GC CAPSULE APPLIERS (1 piece).

\*Only available in limited countries and packages.

## CAUTION

1. In case of contact with oral tissue or skin, immediately remove with a sponge or cotton pellet. After the restorative treatment is finished, rinse thoroughly with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. EQUIA Coat is flammable. Do not use near naked flame. Keep away from sources of ignition. Do not store large quantities in one area.

- Keep away from direct sunlight.
4. EQUIA Coat is volatile. Use in a well ventilated place. Replace cap immediately.
  5. If the tissue contacted by EQUIA Coat turns white or forms a blister, advise the patient to leave the affected area undisturbed, until the mark disappears, usually in 1-2 weeks. To avoid contact, it is recommended to apply cocoa butter to the area where rubber dam cannot cover.
  6. Do not use EQUIA Coat in combination with desensitizers and eugenol containing materials as these may hinder EQUIA Coat from setting or bonding properly.
  7. Avoid inhalation or ingestion of material.
  8. Avoid getting material on clothing
  9. Do not mix with other products.
  10. Instruct the patient not to apply pressure for 1 hour.
  11. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> They can also be obtained from your supplier.

#### Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental) In this way you will contribute to improve the safety of this product.

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