

everStick®C&B

Fibre reinforcement for composite bridges

DISTRIBUTED BY

GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku,
Tokyo 174-8585, Japan

GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240,
Interleuvenlaan 33,
B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

RESPONSIBLE MANUFACTURER IN CANADA

GC AMERICA INC.
3737 W. 127th Street, Alsip, IL 60803 U.S.A.

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1
#06-01 Tampines Plaza
Singapore 529541 TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow,
NSW 2019, Australia
TEL: +61 2 9301 8200

GC SOUTH AMERICA
Rua Heliadora, 399,
Santana - São Paulo, SP, Brasil
CEP: 02022-051
TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TÊC: MAYARA DE SANTIS RIBEIRO -
CRO/SP 105.982



Fibre type: Silanated E-Glass fibre impregnated with Bis-GMA and PMMA

Form: Unidirectional fibre bundle
Diameter: ~1.5-1.7 mm



Keep away from sunlight



Temperature limit

Caution: US Federal law restricts this device to sale by or on the order of a dentist.
NU7610 - 30000524

951328 – NU7610
30000524

everStick®C&B

EN
DE
FR
IT
ES
NL
DA
SV
PT
EL
FI
ZH
ID

FIBRE REINFORCEMENT FOR COMPOSITE BRIDGES
GLASFASERVERSTÄRKUNG FÜR COMPOSITEBRÜCKEN
FIBRE DE RENFORT POUR BRIDGES COMPOSITE
RINFORZO IN FIBRA PER PONTI IN COMPOSITO
FIBRA DE REFUERZO PARA PUENTES DE COMPOSITE
VEZELVERSTERKING VOOR COMPOSITBRUGGEN
FIBERFORSTÆRKNING TIL KOMPOSITBROER
FIBERFÖRSTÄKNING TILL KOMPOSITBROAR
FIBRA PARA REFORÇO DE PONTES DE RESINA COMPOSTA
ΙΝΕΣ ΥΑΛΟΝΗΜΑΤΩΝ ΓΙΑ ΕΝΙΣΧΥΣΗ ΓΕΦΥΡΩΝ ΑΠΟ ΠΟΛΥΜΕΡΗ ΥΛΙΚΑ
LASIKUITULUJITE KUITULUJITTEISIIN YHDISTELMAMUOVISILTOIHIN
用于各种合成牙桥制作的复合树脂纤维
SERAT YANG DIPERKUAT UNTUK JEMBATAN KOMPOSIT



everStick®
C&B 1. A surface retained bridge in the anterior region



