Prior to use, carefully read the instructions for use. **EN**

## FREEGENOL TEMPORARY PACK

EUGENOL FREE TEMPORARY CEMENT

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

Cementation of temporary crowns and bridges.

### CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to rosin.

### DIRECTIONS FOR USE

- Squeeze equal lengths of Base and Accelerator pastes onto the mixing pad.
- Mix thoroughly for 20-30 seconds.
- Insert paste into (temporary) crown or bridge and seat onto the prepared dry tooth.
- Allow to set (3 minutes from start of mixing) and trim excess.
 

Note:

  - Setting time can be adjusted by using more or less Base (More Base=faster set ; less Base=slower set). Do not go outside Base / Accelerator ratio range 3 : 2 to 2 : 3).
  - If material sticks to tooth or metal, Vaseline may be added up to 25% of total mix.

### STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

### PACKAGES

1-1 pkg: 55 g Base, 20 g Accelerator, Cleaner and mixing pad

### CAUTION

- In case of contact with eyes, flush immediately with water and seek medical attention.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS.

Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

or for The Americas

<http://www.gcamerica.com>

They can also be obtained from your supplier.

### CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

### Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)  
In this way you will contribute to improve the safety of this product.

Last revised: 06/2019

**CE 2797**

**'GC'**

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.  
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

RESPONSIBLE MANUFACTURER IN CANADA:

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED by  
GC CORPORATION  
75-1 Hozonuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU** : GC EUROPE N.V.  
Hissarclijgatan Hasselde-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL : +32 16 74 10 00

PRINTED IN JAPAN

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.

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