Prior to use, carefully read the instructions for use. **EN**

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## GC FujiCEM™

RADIOPAQUE REINFORCED GLASS IONOMER LUTING CEMENT  
For use only by a dental professional in the recommended indications.

- RECOMMENDED INDICATIONS**
- Cementation of metal-based inlays, onlays, crowns and bridges
  - Cementation of resin inlays, onlays, crowns and bridges
  - Cementation of all ceramic inlays
  - Cementation of high strength (zirconia based) all ceramic crowns and bridges
  - Cementation of metal, ceramic and fiber posts

- CONTRAINDICATIONS**
- Pulp capping.
  - Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylate monomer or methacrylate polymer.

### IDENTIFICATION OF PARTS (Fig. 1)

- Paste Pak Cartridge
- Cover 2. Inner plunger 3. Cartridge tip
- Paste Pak Dispenser
- Piston 5. Adjusting guide 6. Lever 7. Sliding block 8. Piston release

### DIRECTIONS FOR USE

Paste / Paste ratio (g/g)	2,0 / 1,0
Mixing time (sec.)	10"-15"
Working time (min., sec.) (at 23°C) (from start of mixing)	2'15"
Final finishing commencing time (min., sec.) (after seating the restoration)	4'30"

### PASTE PAK CARTRIDGE LOADING

- Make sure that the piston is completely retracted into the dispenser (Fig. 2).  
Note:  
To retract the piston, push the piston release and at the same time pull the sliding block (on the underside of the dispenser) with fingers.
- Load the cartridge into the Paste Pak Dispenser (Fig. 3).  
Ensure the ▲ mark on the cartridge lines up with the ▲ mark on the Paste Pak Dispenser as illustrated. Then turn the cartridge fully to the right position.
- After cartridge loading, push the piston release forward until it stops (Fig. 4).  
a) Push the piston release forward.  
b) Do not push the upper side.  
Note:  
To remove the cartridge, retract the piston fully and turn the cartridge back to the marked positions.

### CEMENTATION TECHNIQUE

- TOOTH PREPARATION**
  - Prepare tooth in usual manner. For pulp capping, use calcium hydroxide.
  - Clean the prepared tooth with pumice and water.  
Note:  
Use GC Fuji PLUS CONDITIONER on the prepared tooth surface for 20 seconds for optimal adhesion.
  - Prepare thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. DO NOT DESICCATE.  
Prepared surfaces should appear moist (glistening).
- RESTORATION PREPARATION**  
Make sure that the restoration is pretreated and handled according to the manufacturers instructions.
- DISPENSING**
  - Remove the cartridge cover (Fig. 5).
  - Depress the lever to dispense the required amounts of paste onto the mixing pad.  
Note:  
1) Before dispensing the first time from a new cartridge or after a long interval in between use, bleed the pastes from the cartridge.  
2) The adjusting guide can be moved forwards or backwards to change the quantity of material to be dispensed (Fig. 6).  
3) Position of adjusting guide for one inlay.  
4) Position of adjusting guide for one crown.
  - While depressing the lever (Fig. 7-1), level and cut off the extruded material from the cartridge tips by moving the cartridge and dispenser to an upright position on the mixing pad (Fig. 7-2).  
d) When the lever is released after dispensing, residual pastes at the cartridge tips are retracted into the cartridge.  
Clean the cartridge tip and replace the cartridge cover.  
Note:  
The amount of residual material in the cartridge can be controlled by the position of the sliding block on the underside of the dispenser. As the cartridge cylinders are translucent, the residual amount can also be confirmed by the position of the inner plunger seen through the cylinders.

- MIXING**  
After dispensing, spread the two pastes thinly on the mixing pad using a plastic spatula. Mix thoroughly, with lapping strokes, for 10 seconds. Be careful not to incorporate air bubbles.  
Note:  
1) The two pastes are similar in color, so mix with care.  
2) Larger amount of paste requires a longer mixing time (15 seconds).  
3) Do not mix GC FujiCEM pastes with other materials.
- CEMENTATION**
  - Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 2 minutes 15 seconds from start of mixing at 23°C (73°F).  
Higher temperature will shorten working time.  
Refrigeration of the cartridge is recommended to extend the working time.
  - Maintain moderate pressure.
  - Start removing excess cement when the excess cement feels rubbery.
  - Finishing can be started 4 minutes 30 seconds after seating the restoration.

- CLEANING / STERILIZATION OF PASTE PAK DISPENSER**  
For cleaning and sterilization of PASTE PAK DISPENSER, refer to the respective instructions for use.
- SHADE**  
Yellow
- STORAGE**  
Recommended for optimal performance, store in a cool and dark place (4-25°C) (39-77.0°F).  
If not in use for a prolonged period of time, store in refrigerator.
- PACKAGES**
  - GC FujiCEM Refill
  - Paste Pak Cartridge (13.3g / 7.2mL) (2) with mixing pad (No.22)
  - Paste Pak Dispenser (1)

- CAUTION**
- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol.  
Flush with water. To avoid contact, a rubber dam and / or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.
  - In case of contact with eyes, flush immediately with water and seek medical attention.
  - In rare cases metal may cause sensitivity to some people.  
If any such reactions are experienced, discontinue the use of Paste Pak Dispenser (nickel-plated aluminum) and refer to a physician.
  - Store Paste Pak Dispenser away from chemicals such as iodine tincture containing halogen element. Otherwise the dispenser may discolor.
  - GC Fuji PLUS CONDITIONER contains 2% Ferric Chloride.  
It is not recommended to use GC Fuji PLUS CONDITIONER in combination with products containing Epinephrine as this may stain oral tissue.
  - Do not use GC FujiCEM in combination with eugenol containing materials as eugenol may hinder GC FujiCEM from setting or bonding properly.
  - This product is not indicated for filling or core build-up.
  - In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
  - Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:  
<http://www.gc-europe.com>  
They can also be obtained from your supplier.

**CLEANING AND DISINFECTING MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.  
DO NOT IMMERGE: Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects - Reporting:  
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)  
as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)  
In this way you will contribute to improve the safety of this product.

Last revised: 06/2019

Manufactured by GC CORPORATION  
78-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

Distributed by GC CORPORATION  
78-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Researchpark Haastrodde-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium  
TEL: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.  
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541  
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.  
1753 Botany Rd, Banksmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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