Prior to use, carefully read the instructions for use. **EN**

GC Fuji ORTHO™

SELF-CURED ORTHODONTIC BONDING ADHESIVE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Bonding metal brackets and attachments.
- Bonding ceramic brackets.
- Bonding acrylic appliances.
- Band cementation only when extra band retention is desired.

CONTRAINDICATIONS

- Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylate monomer or methacrylic polymer.
- Bonding polycarbonate brackets.
- Cementation of stainless steel crown-retained appliances (e.g., Herbst Appliances).

DIRECTIONS FOR USE

- Enamel Preparation
 - Using a prophyl cup or a brush, clean the bonding surfaces of the teeth with plain (non-fluoridated) pumice and water.
 - Rinse thoroughly with water.
 - Using a sponge or cotton pellet, apply ORTHO CONDITIONER to the bonding surfaces of the teeth for 20 seconds. Rinse thoroughly. Although successful results have been obtained with no enamel pretreatment, conditioning of the enamel bonding surfaces will increase the adhesive's bond strength. It is therefore recommended, especially for new users of GC Fuji ORTHO.
- OPTIONAL:** Etch the enamel bonding surfaces according to the etchant manufacturer's instructions. Rinse thoroughly.

Note: The enamel bonding surfaces must be moist. An overly dry or desiccated enamel surface will adversely affect the bond strength. The optimum level of surface moisture can be obtained by wiping the bonding surfaces of the teeth with a moistened cotton roll immediately prior to bracket bonding. If the bonding surface becomes desiccated during the bonding procedure, rehydrate by wiping them with the moistened cotton roll before bonding with GC Fuji ORTHO.

- Powder and Liquid Dispensing**
 - The standard powder to liquid ratio is 3.0 g/1.0 g, 1 level scoop of powder to 1 drop of liquid.
 - For accurate dispensing of powder, tap the bottle gently. Do not shake or invert.
 - Hold the liquid bottle vertically and squeeze gently.
 - Close bottles immediately after use.

- Mixing**

Divide the powder into 2 equal parts. Mix the first portion with ALL the liquid and mix for about 10 seconds. Incorporate the remaining powder and mix thoroughly for an additional 10-15 seconds (total 20-25 seconds).

Note: The working time is approximately 3 minutes from start of mixing at 23°C (73.4°F). Higher temperatures will shorten the working time, and lower temperatures will extend it.

- Bonding Procedure**
 - Brackets**
 - Coat the bonding surface of the bracket completely with the mixed adhesive.
 - Position the coated bracket on the tooth.
 - Press the bracket firmly against the enamel surface. Using an explorer or a scaler, remove the excess adhesive at this time. Press the bracket again. Continue placing additional brackets. Place all brackets in a quadrant or in the full arch. Note: More than one mix of adhesive may be necessary.
 - AFTER REMOVING EXCESS ADHESIVE, BE CAREFUL NOT TO DISTURB THE BRACKETS BEFORE THE ADHESIVE IS CURED. MOVEMENT OF THE BRACKETS AT THIS POINT WILL DECREASE THE BOND STRENGTH.**
 - Allow the adhesive to cure approximately 5 1/2 minutes from start of mixing.
 - Insert leveling wire and complete initial procedures. A light force wire (e.g., .016 NITI or equivalent) is recommended at the initial bonding.

- Notes:**
 - Bonding ceramic brackets: Chemically-retentive bases must be etched and silanated. If not etched and silanated by the manufacturer, etch and silanate before bonding with GC Fuji ORTHO. If bracket bases become contaminated, follow the manufacturer's directions for cleaning, and/or etching and silanating. No treatment is required for mechanically-retentive bases.
 - Bonding brackets to porcelain restorations: Prepare porcelain surfaces by etching and silanating according to manufacturer's directions. Bond with GC Fuji ORTHO as per above.
 - Bonding brackets to amalgam/metal alloy restorations: Lightly roughen the metal surface with a disc or a fine diamond point before placing GC Fuji ORTHO.

- Acrylic Appliances**
 - Using a microetcher or a rotary instrument, roughen and place retention holes in the internal surface of the appliance.
 - Prepare the teeth as mentioned in section #1.
 - To facilitate removal of the appliance, place petroleum jelly on the occlusal surfaces of the teeth.
 - Fill the internal surfaces of the appliance with GC Fuji ORTHO.
 - Seat into proper position.
 - Immediately remove excessive adhesive from the periphery of the appliance.
 - Allow the adhesive to cure approximately 5 1/2 minutes from start of mixing.
 - Appliance activation can begin at the patient's bedtime the same evening.

- Debonding Procedure**
 - Brackets**

Grasp the bracket with a ligature cutter and, while supporting the tooth with fingers from the lingual side, give a sharp clockwise rotation to the bracket. If resistance is met, try again twisting in the opposite direction after desiccating (using an air syringe) the tooth around the bracket. Remove any residual adhesive with a scaler or rotary instrument.
- Appliances**

To remove bonded appliances, flex the appliance in multiple sites with band removing pliers. If this does not remove the appliance, section the appliance, desiccate the adhesive, and remove the appliance.

- STERAGE**

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39.2-77.0°F).
- PACKAGES**
 - Standard Package: 40 g powder (1), 8 g (6.8 mL) liquid (2) with a powder scoop, a plastic spatula, a set of mixing pads (No. 23)
 - Replacement Package:
 - 40 g powder with a powder scoop
 - 8 g (6.8 mL) liquid

- CAUTION**
 - In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
 - In case of contact with eyes, flush immediately with water and seek medical attention.
 - Do not mix powder or liquid with any other glass ionomer materials.
 - In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
 - Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

- Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas: <http://www.gcamerica.com>
- They can also be obtained from your supplier.
- CLEANING AND DISINFECTING**
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

- Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

Last revised: 06/2019

Rx Only

CE 2797

IGCI

MANUFACTURED BY
GC CORPORATION
1-6-1 Hasegawa-cho, Itabashi-ku, Tokyo 174-8585, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-951-0200
www.gcamerica.com

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