IDENTIFICATION OF PARTS
ОЗНАЧЕНИЕ НА ЧАСТИТЕ
IDENTIFICACIE DILU
PRIKAZ DIJELOVA
AZ ALKATRÁSZEK
MEGNEVEZÉSE
CZEŚCI SKŁADOWE
IDENTIFICAREA
COMPONENTELOR
ПРЕДНАЗНАЧЕНИЕ ЧАСТЕЙ
СИСТЕМА

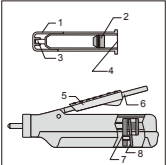
Fig. 1 slika 1 рис. 1 Фиг. 1 1. ábra obr. 1 rys. 1

DIRECTIONS FOR USE
НАСОКИ ЗА УПОТРЕБА
NAVOD K POUŽITÍ
UPUTE ZA UPORABU
HASZNÁLATI ÚTMUTATÓ
INSTRUKCJA STOSOWANIA
INSTRUCȚIILE DE FOLOSIRE
ИНСТРУКЦИЯ ПО
ЭКСПЛУАТАЦИИ

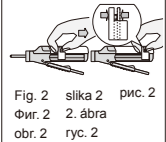
Fig. 2 slika 2 рис. 2 Фиг. 2 2. ábra obr. 2 rys. 2

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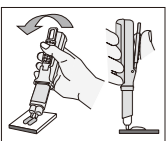
Fig. 7 slika 7 рис. 7 Фиг. 7 7. ábra obr. 7 rys. 7

Paste Pak Dispenser

For use only by a dental professional in the recommended indications.

Paste Pak Delivery System consists of Paste Pak Dispenser and the special Paste Pak cartridge. Paste Pak Dispenser enables you to dispense material easily and correctly from the cartridge.

RECOMMENDED INDICATIONS

Dispensing material from GC special Paste Pak cartridges.

IDENTIFICATION OF PARTS

(Paste Pak Cartridge)
1. Cover 2. Inner plunger 3. Cartridge tip (Paste Pak Dispenser)
4. Piston 5. Adjusting guide 6. Lever
7. Sliding block 8. Piston release (Fig.1)

Before use
1. Remove the fixing tape placed between the main part and lever of the dispenser.
2. Remove the V shaped board from the underside of the dispenser.

DIRECTIONS FOR USE

1. Make sure that the piston is completely retracted into the dispenser. (Fig.2)

Note:
To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them back together.

2. Paste Pak Cartridge Loading. (Fig.3)

a. Ensure the mark on the cartridge lines up with the mark on the Paste Pak Dispenser as illustrated.

b. Then turn the cartridge fully to the right position.

Note:
To remove the cartridge, retract the piston fully and turn the cartridge back to the marked positions.

3. After cartridge loading, push the piston release forward until it stops. (Fig.4)

a. Push the piston release forward.

b. Do not push the upper side.

4. Remove the cover. (Fig.5)

5. a) Depress the lever to dispense the required amounts of paste onto the mixing pad.

Note:
1) Before dispensing the first material from a new cartridge or when dispensing after a long interval, bleed the pastes in order to prevent the incorporation of air bubbles into the material.

2) The adjusting guide can be moved forwards or backwards to change the quantity of material to be dispensed. (Fig.6)
1. Position of adjusting guide for a smaller quantity
2. Position of adjusting guide for a larger quantity

b) While depressing the lever, level and cut off the extruded material by moving the cartridge and dispenser to an upright position on the mixing pad. (Fig.7) When the lever is released after dispensing, residual pastes at the tips are retracted into the cartridge. Replace the cartridge cover.

Note:
1) The amount of residual material in the cartridge can be confirmed by the position of the sliding block on the underside of the dispenser. If the cartridge has translucent tips, the residual amount can also be confirmed by the position of the inner plunger seen through the cartridge tips.
2) Do not drop the dispenser with the cartridge. Otherwise the cartridge may be damaged.

Cleaning / Sterilization of Paste Pak Dispenser

1) Clean the dispenser and adjusting guide with gauze soaked in alcohol.
2) The dispenser may be steam autoclaved at 121-135°C (250-275°F) for 10 to 30 minutes or disinfected using alcohol. (Please remove the adjusting guide and the cartridge before autoclaving the dispenser). The use of various chemical disinfectants / sterilants may cause damage to the dispenser and therefore is not recommended.

Note:
After autoclaving, the piston release of the dispenser may not easily be moved forwards and backwards by fingers. In such a case, slide the piston release forward until it stops. Apply a very small quantity of petroleum jelly onto the entire surface of the piston. Do not apply excess petroleum jelly. Then move the piston release back and forth a few times to ensure smooth movement.

PACKAGE

1 piece

CAUTION

1. In rare cases metal may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product (nickel-plated aluminum) and refer to a physician.
2. Store Paste Pak Dispenser away from chemicals such as iodine tincture containing halogen element. Otherwise the dispenser may discolor.

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