Prior to use, carefully read the instructions for use.

ORTHO GEL CONDITIONER

CLEANING AGENT FOR ORTHODONTIC CEMENTS

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS


ORTHO GEL CONDITIONER is a polyacrylic acid gel solution (approx. 20%) designed to mildly etch and condition enamel, thus enhancing the bond strength of glass ionomer materials to tooth surfaces.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to polyacrylic acid.

DIRECTIONS FOR USE

- Hold the syringe upright and remove the wing cap by turning counterclockwise. Promptly and securely attach the brush tip by turning clockwise.
- Clean tooth surfaces to be bonded with pumice and water. Rinse thoroughly with water.
- Dispense ORTHO GEL CONDITIONER directly onto the tooth surface to be bonded and leave for 10 seconds.
- Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening). Note:
 - When attaching the brush tip, make sure the joint between the tip and the syringe is free of material.
 - After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

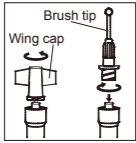STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39,2-77,0°F).

PACKAGE

Syringe (1,4 g/1,2 mL)(3), Brush Tip (25)

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and / or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Brush tip is single use only. Brush tip cannot be sterilized. Reuse or re-sterilization of brush tip could cause patient infection.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas <http://www.gcamerica.com>. They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

Last revised: 05/2021

Rx Only

GC

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC SOUTH AMERICA
Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TEC: Erick de Lima - CRO/SP 100.866

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