GC Fuji ORTHO™ LC PASTE PAK Automix

FOR USE with a dental professional in the recommended indications only.

- RECOMMENDED INDICATIONS**
- 1. Bonding metal brackets and attachments.
 - 2. Bonding ceramic brackets and attachments.
 - 3. Bonding acrylic appliances e.g. Rapid Palatal Expander (RPE).

CONTRAINDICATIONS

1. Do not use in patients with known allergies to glass ionomer cement, methacrylate monomer or methacrylate polymer.

2. Do not use in patients with known allergies to dental acrylic resin.

3. Cementation of stainless steel crown-retained appliances and cast alloy crowns.

- IDENTIFICATION OF PARTS (Fig. 1)**
1. Cover 2. Inner plunger 3. Cartridge tip
 4. Cartridge tip
 5. Cartridge body
 6. Cartridge base

PASTE PAK CARTRIDGE LOADING

Make sure that the piston is completely retracted into the dispenser (Fig. 2).

NOTE:

- To retract the piston, push the piston release and at the same time pull the sliding block on the underside of the dispenser with fingers.
- Load the cartridge into the Paste Pak Automix by pulling the cartridge release.
- Ensure the 'mark' on the cartridge lines up with the 'mark' on the Paste Pak Automix as illustrated. Then depress the cartridge into the right position.
- After cartridge is seated, push the piston release forward until it stops (Fig. 4).
- To retract the piston release forward.
- Do not pull the upper slide.
- To remove the cartridge, retract the piston fully and turn the dispenser back to the retracted position.

DIRECTIONS FOR USE

TOPCH PREPARATION

1. Use the sprays of ORTHO GEL CONDITIONER uniformly and thoroughly to wet the tooth surface. Dry with gauze and securely attach the bracket by lumen conditioning.

CARTRIDGE ACTIVATING WITH GC MIXING TIP FOR LC

1. Rotate the cartridge 1/4 turn clockwise so the lumen is facing upwards.

2. Push the cartridge into the dispenser until it is fully seated.

3. Rotate the cartridge 1/4 turn clockwise so the lumen is facing upwards.

4. Push the cartridge into the dispenser until it is fully seated.

BRACKET PREPARATION

1. Apply the adhesive as pre-treated and hand-dried according to the manufacturer's instructions.

2. Use the GC MIXING TIP FOR LC to mix the adhesive and to apply it to the bracket.

3. Use the GC MIXING TIP FOR LC to mix the adhesive and to apply it to the bracket.

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