Prior to use, carefully read the instructions for use.

## GC Fuji PLUS® CONDITIONER

TOOTH CONDITIONING AGENT

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

GC Fuji PLUS CONDITIONER is a tooth conditioning agent designed for use with GC Fuji PLUS glass ionomer luting cement. It helps to enhance the bond between GC Fuji PLUS and tooth structure.

#### DIRECTIONS FOR USE

- After tooth preparation, apply GC Fuji PLUS CONDITIONER to the bonding surface for 20 seconds using a cotton pellet or sponge.
- Rinse thoroughly with water and dry. DO NOT DESICCATE.
- Proceed according to GC Fuji PLUS directions.
- Close the bottle immediately after use.

#### STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

#### PACKAGES

Bottle of 7 g (6.5 mL)

#### CAUTION

- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- GC Fuji PLUS CONDITIONER contains 2% Ferric Chloride. Not recommended for use with products that contain Epinephrine, which is known to react with iron containing products and may stain oral tissue.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS.

Always familiarize yourself with the safety data sheets available at:

www.gceurope.com or for The Americas

www.gcamerica.com

They can also be obtained from your supplier.

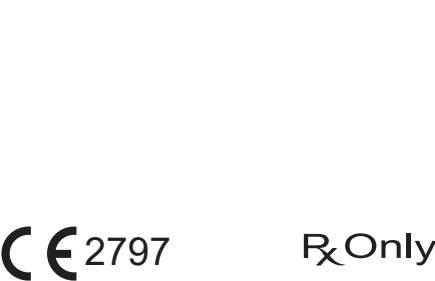
#### CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

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