Prior to use, carefully read the instructions for use. **EN**

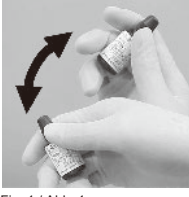
Fig. 1 / Abb. 1

**G-BOND™**

**ONE COMPONENT SELF-ETCHING LIGHT-CURED ADHESIVE**

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**

1. Bonding of light cured composites and acid modified composites (compomers) to tooth structure.
2. Bonding of dual cured luting and core build up composites to tooth structure as long as these materials are light cured.

**CONTRAINDICATIONS**

1. Direct pulp capping.
2. Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer, or acetone.

**NOT TO BE USED**

1. In combination with chemically cured composite resin.
2. In combination with eugenol containing materials as eugenol may hinder G-BOND from setting or bonding properly.
3. In combination with desensitizers as desensitizers may hinder G-BOND from setting or bonding properly.
4. In combination with dual cured luting and core build up composites in case these materials will not be light cured.

Fig. 2 / Abb. 2

**DIRECTIONS FOR USE**

1. CAVITY PREPARATION Prepare tooth using standard techniques. Use rubber dam for protection. Dry the prepared tooth surfaces by blowing with an air syringe.  
Note: For direct pulp capping, use calcium hydroxide.
2. APPLICATION
  - a) Prior to dispensing, shake the bottle of G-BOND thoroughly (Fig. 1).
  - b) Dispense a few drops into a clean dispensing dish (Fig. 2).
  - c) Replace bottle cap immediately after use (Fig. 3).
  - d) IMMEDIATELY apply to the prepared enamel and dentin surfaces using the DISPOSABLE APPLICATOR II (Fig. 4).
  - e) Leave undisturbed for 5-10 seconds after the end of application (Fig. 5).
  - f) Then, dry thoroughly for 5 seconds with oil free air under MAXIMUM air pressure. Use vacuum suction to prevent spatter of the adhesive (Fig. 6). The final result should be a thin, rough, adhesive film with the appearance of frosted glass and which does not visibly move under further air pressure.
  - Note: When removing from refrigerator after long-term storage, leave the bottle of G-BOND to stand at room temperature for several minutes prior to use.
  - 2) Non cut enamel should be treated with a 35-40% phosphoric acid gel, rinsed with water and gently dried before using G-BOND.
  - 3) Apply G-BOND immediately as the material contains a volatile solvent.
  - 4) Remove excess material remaining on the tooth (other than the surfaces to be bonded) using a sponge or cotton pellet as residue removal is hard to remove after light curing.
  - 5) Should the applied material be contaminated with water, blood or saliva prior to light curing, wash and dry the tooth and repeat the procedure by re-applying material.
3. LIGHT CURING Light cure using a visible light curing unit (Fig. 7).  
Irradiation time
 

Halogen / LED (700 mW/cm²)	10 seconds
High power LED (more than 1200 mW/cm²)	5 seconds

In case where the light guide tip is more than 10mm away from the surface to be irradiated, light cure for the following time.

Halogen / LED (700 mW/cm²)	20 seconds
High power LED (more than 1200 mW/cm²)	10 seconds

Note:  
1) Light cure completely for an effective bonding strength.  
2) Use a protective light shield or similar protective eye wear during light curing.

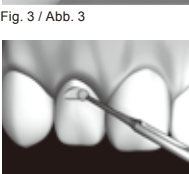
Fig. 3 / Abb. 3

Fig. 4 / Abb. 4

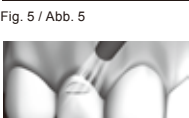
Fig. 5 / Abb. 5

Fig. 6 / Abb. 6

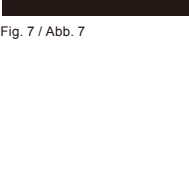
Fig. 7 / Abb. 7

- 4A. PLACEMENT OF LIGHT CURED COMPOSITES AND COMPOMERS After having light cured the adhesive, follow manufacturer's instructions for use in order to place, contour and light cure the composite resin material of choice.
- 4B. PLACEMENT OF DUAL CURED COMPOSITES After having light cured the adhesive, make sure to light cure the dual cured composite material separately. Self curing will result in improper adhesion.
5. FINISHING Finish the restoration with fine finishing burs.  
Adjust restoration and polish using standard techniques.

**STORAGE**  
If not in use for a prolonged period of time, store in refrigerator, otherwise store at room temperature (1-28°C) (33.8-82.4°F).

**PACKAGE**

1. Kit.
  2. mL, liquid (1), dispensing dish (No.1) (1), DISPOSABLE APPLICATOR II (50)
  2. Refill.
  5. mL liquid (1)
- Note:  
All packaging types may not be available in all countries.

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Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:  
http://www.gceurope.com  
or for The Americas  
http://www.gcamerica.com  
They can also be obtained from your supplier.

**CLEANING AND DESINFECTING**  
**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.  
**DO NOT IMMERGE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

**Undesired effects - Reporting:**  
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en  
as well as to our internal vigilance system: vigilance@gc.dental  
In this way you will contribute to improve the safety of this product.

MANUFACTURED BY:  
**GC CORPORATION**  
7F-1 Hasunuma-cho, Habashi-ku, Tokyo 174-8585, Japan  
RESPONSIBLE MANUFACTURER IN CANADA:  
**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60603 U.S.A.  
CNP: #8-271-999/001-01  
DISTRIBUTED BY:  
**GC CORPORATION**  
7F-1 Hasunuma-cho, habashi-ku, Tokyo 174-8585, Japan  
**EU :** **GC EUROPE N.V.**  
Researchpark Hasselde-Lieven 1240, Harelbekestraat 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00  
**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60603 U.S.A.  
TEL: +1-708-587-2360  
www.gcamerica.com

**GC SOUTH AMERICA**  
Rua Helicóps, 395, Santana - 02021-051 São Paulo, SP, Brazil  
TEL: +55-11-2605-0865  
CNP: #8-271-999/001-01  
Responsável Técnica Mayara de Santa Riberio CRO-SIG: 165.882

**GC ASIA DENTAL PTE. LTD.**  
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541  
TEL: +65 6346 7348

**GC AUSTRALASIA DENTAL PTY. LTD.**  
1753 Botany Rd, Bankstown NSW 2191, Australia  
TEL: +61 2 9301 8200

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