Fig. 1 / Abb. 1

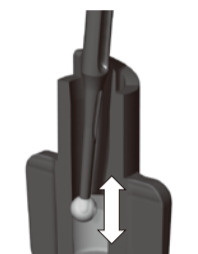
Fig. 2 / Abb. 2

Fig. 3 / Abb. 3

Fig. 4 / Abb. 4

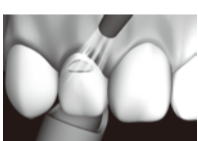
Fig. 5 / Abb. 5

Fig. 6 / Abb. 6

G-BOND™ Unit Dose

ONE COMPONENT SELF-ETCHING LIGHT-CURED ADHESIVE
For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS
1. Bonding of light cured composites and acid modified composites (compomers) to tooth structure.
2. Bonding of dual cured luting and core build up composites to tooth structure as long as these materials are light cured.

CONTRAINDICATIONS
1. Direct pulp capping.
2. Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer, or acetone.

NOT TO BE USED
1. In combination with chemically cured composite resin.
2. In combination with eugenol containing materials as eugenol may hinder G-BOND from setting or bonding properly.
3. In combination with desensitizers as desensitizers may hinder G-BOND from setting or bonding properly.
4. In combination with dual cured luting and core build up composites in case these materials will not be light cured.

DIRECTIONS FOR USE
1. CAVITY PREPARATION
Prepare tooth using standard techniques. Use rubber dam for protection. Dry the prepared tooth surfaces by gently blowing with an air syringe.
Note:
For direct pulp capping, use calcium hydroxide.

2. APPLICATION
a) Unscrew cap counterclockwise (Fig. 1).
b) Using the DISPOSABLE APPLICATOR II provided, mix the luting thoroughly for 5 seconds as illustrated (Fig. 2), making sure to combine liquid from top and bottom of the container.
c) Immediately apply to the prepared enamel and dentin surfaces using the DISPOSABLE APPLICATOR II (Fig. 3).
d) Leave undisturbed for 10 seconds (Fig. 4).
e) Then, dry thoroughly for 5 seconds with oil free air under MAXIMUM air pressure. Use vacuum suction to prevent splatter of the adhesive (Fig. 5). The final result should be a thin, rough, adhesive film with the appearance of frosted glass and which does not visibly move under further air pressure.
Note:

1) When removing from refrigerator after long-term storage, leave the G-BOND Unit Dose to stand at room temperature for several minutes prior to use.
2) When opening the cap, make sure to identify the top and bottom of the unit dose capsule.
3) If the cap of the unit dose is too tight to open, do not try to open it with force. Use another unit dose.
4) Non cut enamel should be treated with a 35-40% phosphoric acid gel, rinsed with water and gently dried before using G-BOND.
5) Apply G-BOND immediately as the material contains a volatile solvent.
6) Remove excess material remaining on the tooth (other than the surfaces to be bonded) using a sponge or cotton pellet as residue material is hard to remove after light curing.
7) Should the applied material be contaminated with water, blood or saliva prior to light curing, wash and dry the tooth and repeat the procedure by re-applying material.

3. LIGHT CURING
Light cure using a visible light curing unit (Fig. 6).
Irradiation time
Halogen / LED (700 mW/cm²) 10 seconds
High power LED (more than 1200 mW/cm²) 5 seconds

In case where the light guide tip is more than 10mm away from the surface to be irradiated, light cure for the following time.
Irradiation time
Halogen / LED (700 mW/cm²) 20 seconds
High power LED (more than 1200 mW/cm²) 10 seconds

Note:
1) Light cure completely for an effective bonding strength.
2) Use a protective light shield or similar protective eye wear during light curing.

4A. PLACEMENT OF LIGHT CURED COMPOSITES AND COMPOMERS
After having light cured the adhesive, follow manufacturer's instructions for use in order to place, contour and light cure the composite resin material of choice.

4B. PLACEMENT OF DUAL CURED COMPOSITES
After having light cured the adhesive, make sure to light cure the dual cured composite material separately. Self curing only will result in improper adhesion.

5. FINISHING
Adjust restoration and polish using standard techniques.

STORAGE
If not in use for a prolonged period of time, store in refrigerator, otherwise store at room temperature (1-28°C) (33.8-82.4°F).


PACKAGE
G-BOND Unit Dose 0.1 mL (50), DISPOSABLE APPLICATOR II (50)
CAUTION

1. G-BOND is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
2. This material is disposable. Do not recap or reuse.
3. In case of contact with eyes, flush immediately with water and seek medical attention.
4. In case of contact with oral tissue or skin, immediately remove with a sponge or cotton pellet. After the restorative treatment is finished, rinse thoroughly with water.
5. If the tissue contacted by the material turns white or forms a blister, advise the patient to leave the affected area undisturbed, until the mark disappears, usually in 1-2 weeks to avoid contact. It is recommended to apply COCOA BUTTER to the area where rubber-dam cannot cover.
6. Avoid inhalation or digestion of material.
7. When spill on the table or floor, immediately wipe off with a dry cloth.
8. Do not mix with other products.
9. Dispose of all waste according to local regulations.
10. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
11. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
12. Do not place under operating light.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gceurope.com or for The Americas
http://www.gcamerica.com
They can also be obtained from your supplier.

Indesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

Last revised: 06/2019

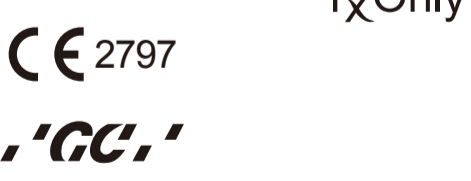MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8588, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL, 60803 U.S.A.
www.gcamerica.com
DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8588, Japan
EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 70 100
GC AMERICA INC.
3737 West 127th Street, Alsip, IL, 60803 U.S.A. TEL: +1 708-597-0900
www.gcamerica.com
GC SOUTH AMERICA
Rua Heliópolis, 369, Santana - 02022-051 São Paulo, SP - Brasil
TEL: +55-11-2925-0965
CNPJ: 08.279.926/0001-41
Responsável Técnica: Mayara de Santos Ribeiro
CRP039 - 105.962
GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541
TEL: +65 6461 7588
GC AUSTRALASIA DENTAL PTY. LTD.
1753 Bolony Rd, Bankmeadow, NSW 2019, Australia
TEL: +61 2 9301 8200
PRINTED IN JAPAN

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In case where the light guide tip is more than 10mm away from the surface to be irradiated, light cure for the following time.
Belichtungszeit
Halogen / LED (700 mW/cm²) 20 Sekunden
Hocheleistungs-LED (über 1200 mW/cm²) 10 Sekunden

Note:
1) Light cure completely for an effective bonding strength.
2) Use a protective light shield or similar protective eye wear during light curing.

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In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
12. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
13. Non posizionare sotto la lampada del riunito.

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