Prior to use, carefully read the instructions.

## GC Initial Connector Paste

A ceramic paste newly developed for base firings on refractory die materials.

### RECOMMENDED INDICATIONS

Base firing on refractory die materials, like G-Cera Orbit Vest.

### DIRECTIONS FOR USE

- Prepare and degas "refractory die model" following manufacturers IFU.
- Stir the Connector Paste before use.
- Apply a very thin layer of the paste to the dry refractory die material. The refractory die material should be totally dried before application.
- The "Connector Paste" coating should extend slightly over the preparation line.
- Appearance after firing should be clear and shiny.
- In case a whitening effect appears after firing, a too thick layer of Connector Paste has been applied.

### FIRING PROGRAM

Pre-heating t°	Drying time	T° increase	Vacuum	Final t°	Holding time
450°C	10 min	60°C/min	Yes	950°C	1 min no vacuum

### STORAGE

Recommended for optimal performance, store in a cool, dry place.

### NOTES

- Always use a clean brush, specially chosen for Connector Paste only.
- GC Initial Connector Paste has been developed to perfectly match the build up ceramics of GC Initial MC and GC Initial LF.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

### PACKAGES

GC Initial Connector Paste, 6g

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas <http://www.gcamerica.com>. They can also be obtained from your supplier.

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: <mailto:vigilance@gc.dental> or for The Americas <mailto:vigilance@gc.dental>. In this way you will contribute to improve the safety of this product.

Last revised: 07/2019

MANUFACTURED and DISTRIBUTED by

EU : GC EUROPE N.V.

Researchpark Haardse Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

RESPONSIBLE MANUFACTURER IN CANADA: GC AMERICA INC. - 3737 W. 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY: GC AMERICA INC. - 3737 West 127th Street, Alsip, IL 60803 U.S.A. TEL: +1-708-597-0900 - [www.gcamerica.com](http://www.gcamerica.com)

GC ASIA DENTAL PTE LTD. - 5 Tampines Central 1 #06-01 Tampines Plaza, Singapore 529541-TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY LTD. - 1753 Botany Rd Bankmeadow, NSW 2019, Australia - TEL: +61 2 9301 8200

GC SOUTH AMERICA - Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL CEP: 02022-051 - TEL: +55-11-2925-0965, CNPJ: 08.279.999/0001-61 RESP: TEC. Mayara de Santos Ribeiro - CRO/SP 105.962

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