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September 2005

S 6100

GB Specially new developed Ti Bonder (Powder & Liquid) to secure optimal bonding between Titanium frame structures and Initial Ti porcelain.

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'GC'
FIRST IS QUALITY

GB - Instruction for Use

Prior to use, carefully read the instructions.

RECOMMENDED INDICATIONS

- Crown & Bridge structures made of pure Titanium or a Titanium alloy.
- For casted Titanium as well as for milled Titanium.

DIRECTIONS FOR USE

- Finishing the frame structure is carried out with staggered tothing burs. These burs should only be used for grinding Titanium.
 - Grind with low pressure and slow speed (approx. 15000 rpm)
 - Frame work should be grinded in one direction only
 - Sharp edges and overlapping should be avoided
- Sandblast with noble corundum only.
 - Grain size 120 – 150 µm / 2 bar pressure
 - Blast in an angle of 45°C to the object
 - Steam clean the surface
 - Surface needs a passivating time of 5 minutes
- Mix the GC Initial Ti Bonder powder with the special liquid in a very thin consistency and apply the mixture in a very thin layer. Do not apply too thick, the coping should shine through.
- Pre-heating temperature: 450°C
Drying time: 4 min
Raise of temperature: 55°C/min
Vacuum: Yes
Final temperature: 810°C
Holding time: 1 min, no vacuum
- The fired bonder surface is dark and has a slightly shining appearance. Thicker areas are slightly gray, this should be avoided.

STORAGE

Store in a cool, dry place.

SHELF LIFE

GC Initial Ti Bonder Powder: 10 years
GC Initial Ti Bonder Liquid: 4 years

NOTES

- Clean the grinding burs using a steam cleaner.
- The Ti Bonder layer does not replace the wash layer.

PACKAGES

GC Initial Ti Bonder Powder, 4gr
GC Initial Ti Bonder Liquid, 25ml
GC Initial Ti Bonder Intro Kit (Powder 4gr + Liquid 25ml)

CAUTION

In case of contact with eyes, flush immediately with water and seek medical attention.

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use and refer to a physician.

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