

NEW CASTING LINER

NEW CASTING LINER is a dry, fiber-type ceramic casting liner designed specially for phosphate-bonded investments.

FEATURES

1. An optimum cushion always helps to produce precision castings.
2. Helps to eliminate cracks in the investment, due to improved heat resistance during burnout.
3. Easy to place in casting rings, due to the optimal flexibility.

USAGE

1. Refer to the chart. Cut the casting ring which allows more than 5mm space between the wax pattern (or resin pattern) and the surface of the casting liner.

	Thickness (mm)	Width (mm)		
	1.0	55		

PACKAGE

Box of 20m roll

MANUFACTURED by

GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by

GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC EUROPE N.V.

Research-Park, 13 Interleuvenlaan, B-3001, Belgium

GC AMERICA INC.

3737 West 127th Street, Alship, IL. 60803 U.S.A.

GC ASIA DENTAL PTE. LTD.

19 Loyang Way, #06-27 Singapore 508724