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Prior to use, carefully read the instructions for use.

## EN

## REBARON

SELF-CURING ACRYLIC FOR DIRECT REBASING

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**
For the rebasing (relining) of acrylic dentures.

**CONTRAINDICATIONS**
Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

- DIRECTIONS FOR USE**
- PREPARATION
    - Check the fit in the mouth using FIT CHECKER (II) or a thinly mixed alginate impression material.
    - Mark the boundary between thick and thin layers using a round bur.
    - Clean the contact surface thoroughly using pumice and water and abrade the area to be rebased. (Pressure relief holes may be drilled in areas of extensive undercuts).
    - Protect the tooth and gingival areas of the denture with COCOA BUTTER, and apply local anaesthetic or glycerine to the patient's mucous membrane contact areas to prevent irritation.
  - MIXING
    - The standard powder/liquid ratio is 1.5 g/1.0 mL, (half a measuring cup of powder to 1/3 of a syringe of liquid).
Note: Excessive quantities of liquid can cause strong irritation.
    - Measure into the jar and mix for 20-30 seconds.
  - REBASING
    - 30 seconds after mixing, coat the mixture onto the denture and leave until the mixture has lost its sheen (1 minute 30 seconds).
    - When the surface is dull, insert the denture into the mouth and make muscle trimming in the centric occlusal position.
    - Wait at least 2 minutes before carefully removing the denture. Correct the periphery.
    - Re-insert the denture in the mouth, and retain it in the centric occlusal position for about 2 minutes.
Note: If the heat generated causes the patient discomfort, allow the denture to cure outside the mouth.
    - REBARON will cure completely in about 9 minutes from start of mixing. Perform final correction and polishing using standard techniques.

**STORAGE**
Recommended for optimal performance, store in a cool place away from direct sunlight (4-25°C) (39.2-77.0°F).

**SHADES**
No. 3 - Pink
No. 8 - Live pink
Pink X

**PACKAGES**

- Bottle of 100 g powder
- Bottle of 100 g (104 mL) liquid
- 1-1 package: 100 g powder, 100 g (104 mL) liquid, plus accessories

Note:
All packaging types may not be available in all countries.

**CAUTION**

- The liquid is volatile and highly flammable. Close the bottle immediately after use and keep away from sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>
They can also be obtained from your supplier.

**CLEANING AND DISINFECTING:**
**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
**DO NOT IMMERSE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)
as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)
In this way you will contribute to improve the safety of this product.

Last revised: 07/2019

**GC**

MANUFACTURED by:
**GC DENTAL PRODUCTS CORP.**
2-285 Toriimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by:
**GC CORPORATION**
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU: GC EUROPE N.V.**
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

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