## GC RELINE™ PRIMER R

RESIN PRIMER/ADHESIVE FOR VINYL POLYSILOXANE DENTURE LINER

## GC RELINE™ MODIFIER

COATING MATERIAL FOR VINYL POLYSILOXANE DENTURE LINER

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- GC RELINE™ PRIMER R : Bonding of GC RELINE™ Soft or Extra Soft and MODIFIER to acrylic denture resin.
- GC RELINE™ MODIFIER : Coating of edges of GC RELINE™ Soft or Extra Soft and acrylic denture resin.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.


### DIRECTIONS FOR USE : GC RELINE™ PRIMER R

- Check and adjust the resin denture in normal manner.
- Relieve the area of the denture to be relined.
- Clean the denture with detergent and water and dry thoroughly. Apply GC RELINE™ PRIMER R (resin primer) to the bonding surface with a clean, dry brush and gently dry with clean, dry oil-free air. Do not contaminate the surface with latex gloves, etc. Note : Do not contaminate the bottle of PRIMER R with moisture. Otherwise the primer liquid may turn cloudy. Replace the cap immediately after use.
- Dispense GC RELINE™ Soft or Extra Soft material directly onto the denture. Seat the denture in the mouth for muscle trimming. Then remove it from the mouth and then progressively trim and polish the edge of the GC RELINE™ material. (See full instructions for use in GC RELINE™ Soft or Extra Soft package.)

### DIRECTIONS FOR USE : GC RELINE™ MODIFIER

- If considered necessary to achieve an even smoother finish , apply PRIMER R to the peripheral borders and margins of the denture. Do not apply the primer to the GC RELINE™ Soft or Extra Soft material.
- After gently drying the primer(as in 3 above), apply the mixture of GC RELINE™ MODIFIER A and B to the edges of the previously placed GC RELINE™ material and the denture. (See full instructions for use in GC RELINE™ Soft or Extra Soft package.)

### Illustrated technique

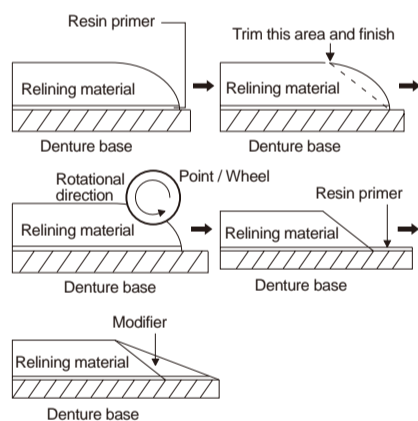### STORAGE

Store at room temperature 15 - 25°C (59 - 77°F) out of direct sunlight. (Shelf life: 2 years from date of manufacture)

### PACKAGES

- GC RELINE™ PRIMER R Primer 12g (13mL)
- GC RELINE™ MODIFIER 1-1pkg. A - 10g (9mL), B - 10g(9mL), Mixing pad (No.15), Sponge

### CAUTION

- The primer is volatile and flammable. Do not use it near fire. Close the bottle immediately after use and keep away from sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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