Prior to use, carefully read the instructions for use.

SOFT-LINER

SOFT DENTURE RELINE MATERIAL

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For the temporary rebasing (relining) of acrylic dentures. Tissue conditioning.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate polymer, ethanol or phthalic ester-based plasticizer.

DIRECTIONS FOR USE

1. PREPARATION
 - a) Clean the contact surface thoroughly using pumice and water and abrade the area to be relined.
2. MIXING
 - a) The standard powder / liquid ratio is 2.2 g / 1.8 g, (first graduation on powder measure to 4 graduations of the liquid syringe). Note: Consistency may be varied by changing powder / liquid ratio.
 - b) Measure into the glass jar and mix for 30-60 seconds.
3. RELINING
 - a) Coat the mixture onto the denture as evenly as possible.
 - b) Insert the denture in the mouth, muscle trim and retain it in the centric occlusal position for 4-5 minutes.
 - c) Remove and inspect for defects.
 - d) Dip the denture into cold water, and trim away excess at the periphery with a heated sharp instrument.

STORAGE

Recommended for optimal performance, store in a cool dark place away from direct sunlight (4-25°C) (39.2-77.0°F).

PACKAGES

1-1 package: 100 g powder, 100 g (97 mL) liquid, plus accessories

NOTE

Close bottles immediately after use.

CAUTION

1. In case of contact with eyes, flush immediately with water and seek medical attention.
2. Do not mix with other products.
3. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

Last revised: 01/2020

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
TEL: +65 6546 7589

EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 5295541
TEL: +65 6546 7589

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow, NSW 2019, Australia
TEL: +61 2 9301 8200

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