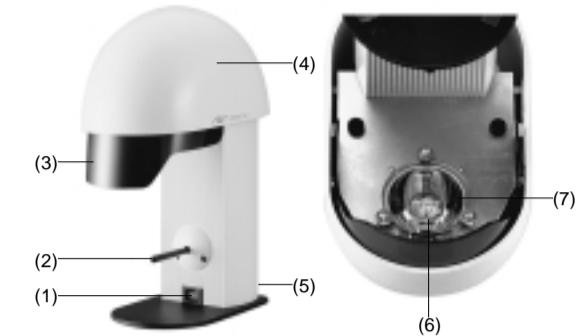
## STEPLIGHT SL-I

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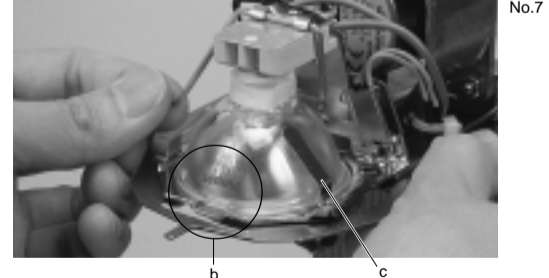
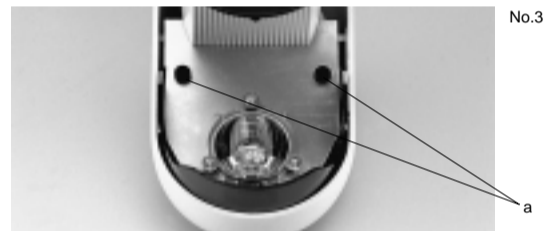
## STEPLIGHT SL-I PRE-CURING LIGHT UNIT

For use only by a dental professional as recommended in these instructions.

### RECOMMENDED USE

Laboratory based pre-curing of visible light-cured dental composites (excluding opaque materials).

### CAUTION

- 1) STEPLIGHT SL-I is not intended to be used for final curing.
- 2) Indoor use only.
- 3) Only use an electrical outlet with a ground (earth) connection.
- 4) Locate the unit in a clean, well ventilated, vibration free area away from direct sunlight or other sources of heat.
- 5) When cleaning or changing parts always unplug the unit.
- 6) Do not look directly at the lamp while it is illuminated.
- 7) Do not touch the lamp or heat absorbing filter during use or for a time after turning off to prevent burns.
- 8) If damaged, replace the protective shield before using.

### IDENTIFICATION OF PARTS

- (1) Main Switch / Pilot Lamp  
With the main switch on, the pilot lamp illuminates.
- (2) Irradiation Switch  
Push "up" or "down" gently to illuminate the lamp.
- (3) Visor
- (4) Head Cover
- (5) Power Cord
- (6) Halogen Lamp
- (7) Heat-absorbing Filter (Protective Shield) to cut infrared rays

### DIRECTIONS FOR USE

1. Plug in the unit.
2. Turn on the main switch. The pilot lamp illuminates.
3. Place the prosthesis in the "curing spot" beneath the lamp under the visor and activate by pressing the irradiation switch.  
Notes  
1) The lamp goes out automatically 10 seconds after releasing the irradiation switch. While the lamp is on, the cooling fan runs continuously.  
2) If the main switch is turned off during irradiation, wait for a few seconds before turning on again. Otherwise the life of the lamp will be adversely affected.  
3) If the prosthesis is located outside the curing spot, curing performance will be reduced.
- 4) In periods of continuous use, the lamp may not illuminate or may go out automatically. This is to prevent the lamp burning out due to an excessive rise in temperature. In such a case, allow it to cool first and then switch on again.

### REPLACEMENT OF HALOGEN LAMP

- Always use the recommended GC special lamp for the GC STEPLIGHT SL-I. Use of any other lamp may result in ineffective curing of the resin or a breakdown of the unit.
1. Turn off the main switch and unplug the unit.
  2. Make sure that the lamp to be replaced is cool.
  3. Remove the two screws (a) on the underside of the head cover and lift it off.
  4. Pull the screw on the side of the lamp cover and lift it off.
  5. Remove the visor.
  6. Pull the visor wide at both ends to detach it from the hooks and pull it towards you to remove.  
Note: Opening the visor too wide may result in damage.
  7. Insert a new lamp.  
Handle a new lamp with care. Wear gloves or use a dry cloth so as not to contaminate it with your fingers. When fitting the lamp, push and slide it fully into the socket so that the convex portion (b) on the lamp is at the front.
  8. In reverse order of disassembly, mount the visor, lamp cover and head cover.  
Note: Mount the head cover with the product name to your right when looking from the front.

### CARE OF THE STEPLIGHT SL-I

1. Do not attempt to modify the unit.  
When cleaning, always switch off and unplug.  
Check the unit periodically to ensure satisfactory operation.
2. Clean the heat-absorbing filter when it gets dirty or curing performance may be affected.  
To clean the heat-absorbing filter and the unit exterior, soak a soft cloth or gauze in lukewarm water containing a little detergent, wring well and wipe.  
Avoid using strong solvents such as thinner or petroleum benzene.

### TROUBLESHOOTING

In case of difficulty, check the following before seeking repair:

Symptom	Check	Correction
The pilot lamp does not come on even if the main switch is turned on.	Check if the plug is disconnected.	Plug in correctly.
The lamp does not come on even if the irradiation switch is turned on.	Check if the lamp is dead. Check if the lamp is disconnected from the socket.	Replace the lamp with a new one by referring to "Replacement of the lamp" mentioned as above. Push and slide the lamp fully into the socket.
	Check if the unit has been used continuously for a long period.	The lamp protection circuit is working. Cool the unit for a while before reuse.

### WARRANTY / REPAIR

Warranty : 1 year from the date of purchase.

In case of a breakdown during this term, repair will be carried out free of charge provided that the device has been used under normal conditions and according to the instructions for use.  
Note: Consumables (such as lamps) are not guaranteed.

### SPECIFICATIONS

Power supply	AC120V 60Hz	AC220 / 230V 50Hz
Power consumption	180W 1.7A	
Lamp rating	AC100V 150W	
Effective room temperature	10-30°C (50-86°F)	
Heat-absorbing filter (Protective Filter)	Specific glass 50mm (dia.) x 4mm (thick.)	
Dimensions	115mm(W) x 210mm (D) x 275mm(H)	
Weight	2 kgs.	

Specifications are subject to change.

Prior to use, carefully read the instructions for use.

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