Prior to use, carefully read the instructions for use.

TEMPRON

TEMPORARY CROWN AND BRIDGE RESIN

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For the preparation of temporary crowns, bridges or dentures. Space maintenance.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

- A. CROWNS**
- PREPARATION**
 - Fill defects in the tooth with CAVITON or equivalent temporary filling material.
 - Take an impression using a heavy (i.e. less water) alginate impression material. Wrap the impression in wet gauze to prevent distortion.
 - Prepare the tooth to be crowned.
 - Cut V notches in the impression from gingival to buccal and lingual sides to allow escape of excess resin.
 - MIXING**
 - The standard powder/liquid ratio is 1.0 g/0.5 mL (up to the first graduation on the powder measure and the first graduation of the liquid pipette).

Note:

To minimize shrinkage use as little liquid as possible.
 - Measure the liquid into the mixing jar, and slowly add the corresponding quantity of powder. Mix for 20-30 seconds.
 - APPLICATION**
 - Pour the creamy paste into the prepared impression, and seat tightly in the mouth.
 - After 3 minutes in the mouth remove the impression, leaving the dough like resin on the tooth.
 - Remove the resin crown before it is fully set and dip in and out of hot water (approximately 50°C/122°F) several times in order to prevent shrinkage.
 - Trim any excess using a bur and then polish.
 - Cement the temporary crown in position using FREEGENOL TEMPORARY PACK or other appropriate temporary cement.
- B. BRIDGES**
- PREPARATION**
 - Select a suitable artificial tooth and fix between the abutment teeth using a suitable wax. Check the bite.
 - Take a full arch impression using an alginate impression material. Carefully remove the pontic and wrap the impression in wet gauze.
 - Prepare the abutment teeth as appropriate.

Subsequent procedures are carried out as described above for crowns A. 1 (a) to A. 3 (e).

STORAGE
Recommended for optimal performance, store in a cool place away from direct sunlight (4-25°C) (39.2-77.0°F).

PACKAGES

- Bottle of 100 g powder
- Bottle of 100 g (104 mL) liquid
- 1-1 package: 100 g powder, 100 g (104 mL) liquid, accessories

CAUTION

- The liquid is volatile and highly flammable. Close the bottle immediately after use and keep away from sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Use of polysulfide rubber based impression materials is not recommended.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

http://www.gceurope.com

They can also be obtained from your supplier.

CLEANING AND DISINFECTING:
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

Last revised: 07/2019

'GC'
GC

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by
GC CORPORATION
6-1 Hatanuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU
GC EUROPE N.V.
Researchpark Haasrode-Luven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Bankmeadow, NSW 2019, Australia
TEL: +61 2 9301 8200

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